

ClearClick® Portable Digital Magnifier

Quick Start Guide & User's Manual



www.ClearClick.tech

Full 1-Year Warranty & FREE USA-Based Tech Support:

support@clearclick.tech

Thank You

Thank you for your purchase of the ClearClick® Portable Digital Magnifier (Model PM18)! We appreciate your business.


At ClearClick®, we try to make our technology as clear to understand and use as possible. But if you get stuck at any point, you qualify for FREE USA-based tech support as a ClearClick® customer. Just email us at: support@clearclick.tech.

To get started with your ClearClick® Portable Digital Magnifier, please see the quick start guide on the following pages.

Safety Precautions

- 1) Do not disassemble the unit.
- 2) Do not operate the unit around water or rain.
- 3) Do not operate the unit around any kind of machines or appliances (other than computers).
- 4) Turn off and unplug the unit when not in use.
- 5) Do not allow young children to use this device.

What's In The Box?

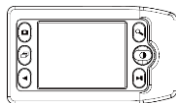

Portable Digital Magnifier



USB Cable




Cleaning Fabric



User's Manual




Carrying Bag

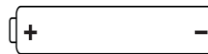


AAA Alkaline Batteries

Quick Start Guide

1. Load 3 AAA alkaline batteries into the device by slightly pushing and pulling the battery door latch to open the battery compartment. Load the batteries according to the polarities shown inside the battery compartment, then close the battery door. Do not mix old and new batteries during battery replacement.

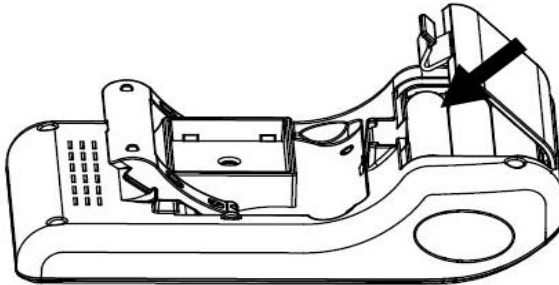

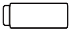When the battery is low, there will be a low-power indicator  shown on the lower right corner of the LCD screen.

When replacement of new batteries is needed, use only Alkaline AAA size batteries. Do not mix old and new batteries during battery installation.


2. Turn on the unit by pressing the Power button located on the side of the device.


3. Pull out the “foot” of the Magnifier until it sits at its maximum range.

4. Place the device flat against the text you want to read.

5. Slide the unit gradually to the right and left to read each


line of text.

6. To cycle through the levels of magnification, **press the Magnify button**  .

7. To change the colors used on the screen, **press the Color Mode button**  until you find the mode best suited for your vision needs.


8. Turn off the unit by pressing the Power button when not in use.

How To Change The Color Mode

Press the Color Mode button  to cycle through the Magnifier's 3 different color modes as follows:

- ◆ Full color
- ◆ Enhanced positive image (black on white)
- ◆ Enhanced negative image (white on black)

How To Increase or Decrease the Magnification Level

Press the Magnify button  to cycle through the four levels of magnifications: 5X, 6X, 7.5X, or 10X.


How To Freeze/Unfreeze The Image

Press the Freeze/Unfreeze  button to switch between




freezing and unfreezing the current magnified picture.


How To Capture Digital Photos

Load a micro SD card (not included) into the micro SD card slot.

Press Snapshot  button to capture a photo. The captured photos will be automatically stored in the micro SD card.

How To Play Back Captured Images

Press the Playback  button to view captured photos. Press the Backward  and Forward  buttons to move forward and backward through the images again.

To return to “Reading” mode, press the Playback  button again.


How To Download Captured Pictures to Your PC or Mac

Connect the Magnifier to a PC or Mac by USB Cable to a USB port on your computer. Turn the Magnifier on. Browse the new drive which will appear on your computer, browse the folders and find your captured images.

How To Turn On/Off The Device

Turn on and off the unit by pressing the Power button located at the side. To save battery power, please always turn off the Magnifier when not using it. The Magnifier will turn off automatically if it is not in use after 5 minutes.

Part Names & Functions

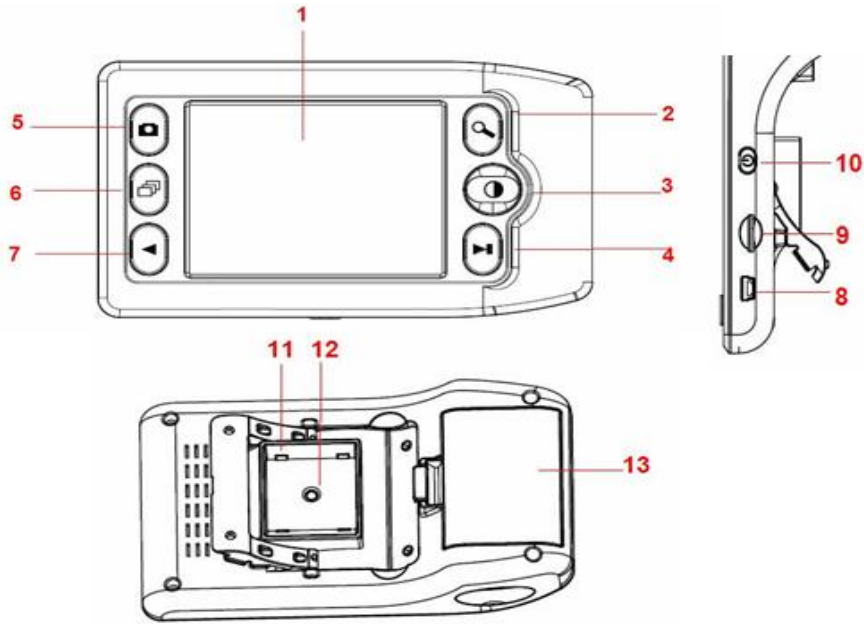

(See the next page for explanations of each part.)

1. 3.5-inch LCD
2. Magnify button
3. Color Mode
4. Freeze/Unfreeze/
Backward
5. Snapshot
6. Playback
7. Forward
8. USB port
9. Micro-SD card slot
10. Power on/off
11. LED
12. Lens
13. Battery
Compartment

Troubleshooting

Problem	Solution
The screen is blank.	Ensure that the power is on. Ensure the battery has enough power. Verify that the item you want to magnify is positioned correctly under the center of the screen

The unit does not turn on.	Change the battery with new ones. Check if the batteries are installed in the right direction.
The image on the screen is dark and/or distorted.	Ensure that the unit is positioned flat on the page you are reading. Ensure that the foot of the magnifier flips out to its maximum position.
There are smudges or blemishes on the screen	Clean the screen and the Lens (located on the bottom of the unit).
Text is too small to read	Press the magnify button to increase magnification.

**Need Help? Email Us! Full 1-Year Warranty & FREE
USA-Based Tech Support: support@clearclick.tech**

This symbol on the product or in the instructions means that your electrical and electronic equipment should be recycled at the end of its life at an appropriate electronics recycling facility.

